

SquintScope®

THE EYE FIXATION DEVICE

Player instructions



Turn ON

Press button on right side to turn on

Swipe up

Press Gallery

Click on the video

Click Play (▶)

(You may pause by pressing the video and then press Pause (⏸),
or use the remote to pause)

Insert the player in SquintScope.

Turn Off

Remove the player from SquintScope.

Press button on right side for a few seconds (if this turns on the screen,
repeat)

Press "Power off"

Remote instructions



General

Press ⏪ to start or pause the video

Press + to increase the Volume

Press - to decrease the Volume

Press ⏪ to start the video

If the player doesn't start, the player has turned the screen off

Press any button for a few seconds (turns on the screen)

Press ⏪ for a few seconds (starts the player)

Press ⏪ to start the video

You may press ⏹ for a few seconds to stop the player and turn off the
screen.

If the blue light flashes for a short while, it shows
that the Bluetooth connection is being re-established.

Repeat the button press if needed.

